


## Equipment Layout

EXISTING CEILING HEIGHT 9'-7"  
 MINIMUM CEILING HEIGHT 9'-3"  
 RECOMMENDED CEILING HEIGHT 9'-6"

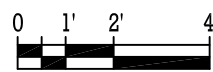

09225-A1

## EQUIPMENT LEGEND

CFCI.....CONTRACTOR FURNISHED AND CONTRACTOR INSTALLED  
 OFCI.....OWNER FURNISHED AND OWNER OR CONTRACTOR INSTALLED  
 OFOI.....OWNER FURNISHED AND OWNER OR VENDOR INSTALLED

	ITEM	DESCRIPTION	SIZE	WT/LBS	BTU/HR	DETAIL SHEET
OFOI	①	QUANTUM ODYSSEY SERIES CONTROL CONSOLE	18" W 12" D 3" H	5	N/A	
OFOI	②	QUANTUM ODYSSEY SERIES GENERATOR	24" W 20" D 34" H	431	4000	
OFOI	③	QUANTUM QV-800 UNIVERSAL SYSTEM	83" W 67" D 103" H	1700	873	
OFOI	④	QUANTUM QT-711 PATIENT TABLE	85" W 28" D 27" H	105	N/A	
OFOI	⑤	CANON DR SYSTEM TOUCH SCREEN OPERATOR CONTROL AND KEYBOARD	18" W 6" D 20" H	26	310	
OFOI	⑥	CANON DR SYSTEM CONTROL PC CABINET (UNDER COUNTER)	4" W 12" D 14" H	77	854	
OFOI	⑦	CANON CXDI 40/50 SENSOR POWER SUPPLY UNIT	14" W 3" D 8" H	10	100	
OFOI	⑧	CANON CXDI 50 SERIES IMAGING UNIT	12.8" W 12.9" D 0.8" H	N/A	N/A	
OFOI	⑨	CANON DR SYSTEM HUB (UNDER COUNTER)	6" W 5" D 2" H	2	100	
	⑩	EXISTING LEAD GLASS VIEW WINDOW	N/A	N/A	N/A	
CFCI	⑪	COUNTER TOP (30-40" AFF)	24" D	N/A	N/A	

CUSTOMER:

THESE DRAWINGS ARE SUBMITTED TO SUGGEST LOCATION OF ALL DIAGNOSTIC IMAGING EQUIPMENT AND ASSOCIATED APPARATUS, ELECTRICAL WIRING DETAILS, AND ROOM ARRANGEMENTS. IN PREPARING THIS DRAWING, EVERY EFFORT HAS BEEN MADE TO CONFORM DETAILS TO THE ACTUAL EQUIPMENT EXPECTED TO BE INSTALLED. HOWEVER, THESE DRAWINGS ARE NOT TO BE USED FOR ACTUAL CONSTRUCTION PURPOSES, AND THE PREPARER CANNOT ACCEPT ANY RESPONSIBILITY FOR DAMAGES RESULTING THEREFROM. CONSTRUCTION DRAWINGS TO BE PREPARED BY PURCHASER'S ARCHITECT OR CONTRACTOR.



DRAWN BY:

R. Rider

DATE:

10/05/09

DWG. NO.

09225

SHEET:

A1

Equipment Layout / Legend